**MODEL: UN 600A**

**ULTRASONİK NEBULİZATÖR**

**“ULTRASONIC NEBULIZER / USER GUIDE“**

**CE** 1984

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Ortam nemlendirme

## 1. GENERAL INFORMATION

NEBTIME ultrasonic nebulizer devices are produced to ensure the use of liquids in aerosol and steam treatment.

In the transformation of liquids into steam form instead of heating method frequency is applied. Frequencies formed by microprocessors electronic system by means of piezo-ceramic crystal are transformed into mechanical vibrations. By means of those vibrations, liquid preserved in the pot is evaporated without heating. Moreover, a fan located in the device absorbs air from the external environment. That air being absorbed passing along a bacterium filter is purified and pressed into the liquid reservoir. The steam which is continuously formed in the liquid container is pushed out with this air pressure and delivered to the patient by means of the hose. That composed steam can be heated by a heater at the outlet.

NEBTIME ultrasonic nebulizer is manufactured to enable practical use with microprocessor control

128\*64 LCD

Control panel and key pad

Visual and audio warning systems

Adjustable steam power intensity, fan speed, timer, heater level

Liquid feeding unit

Plastic body

Bacterium filter

Low energy consumption

Easy to replace Piezoceramic crystal

Outlet steam heater

Drug containing implementation.

At the same time single component foot system applied in the devices provide practical installation.

## 2. FIELDS OF USE

Moisturizing respiration and narcotic gases.

Acute and chronic bronchitis

Bronchial asthma

Pneumonia

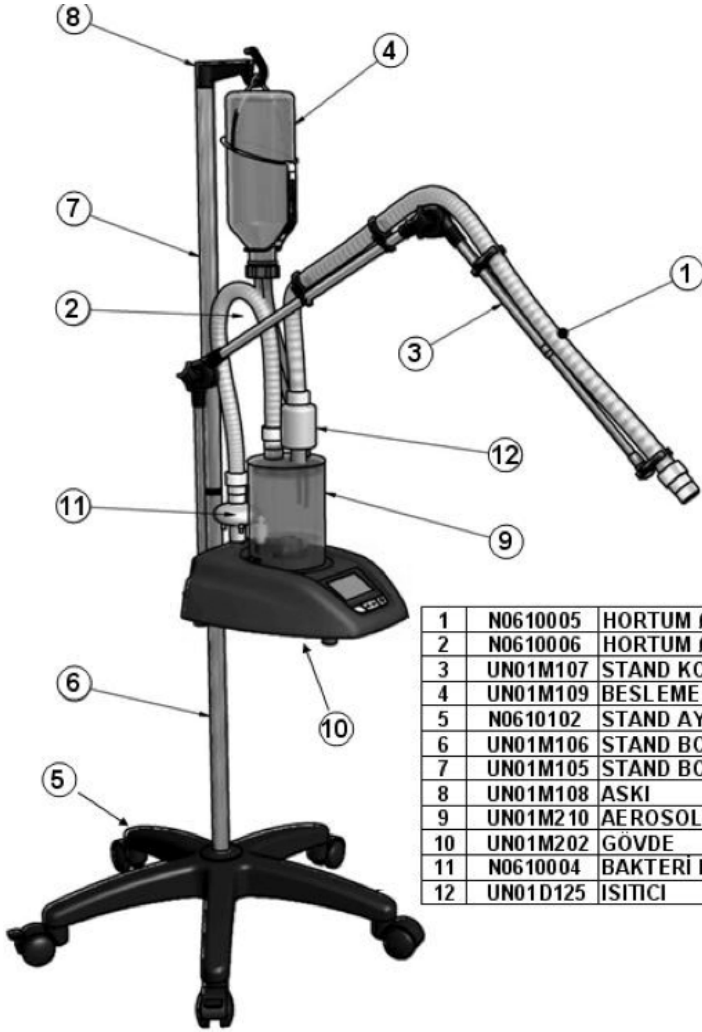
Specific and non-specific lung diseases

Pre-bronchoscopy local anesthesia

Drug and water nebulization


Ambient humidification

### 3. CİHAZ MONTAJ RESMİ VE YEDEK PARÇA LİSTESİ

|    |          |                    |
|----|----------|--------------------|
| 1  | N0610005 | HORTUM Ø20x1200    |
| 2  | N0610006 | HORTUM Ø20x450     |
| 3  | UN01M107 | STAND KOLU         |
| 4  | UN01M109 | BESLEME ŞİŞESİ     |
| 5  | N0610102 | STAND AYAGI TAKIMI |
| 6  | UN01M106 | STAND BORUSU ALT   |
| 7  | UN01M105 | STAND BORUSU UST   |
| 8  | UN01M108 | ASKI               |
| 9  | UN01M210 | AEROSOL KAVANÖZ    |
| 10 | UN01M202 | GÖVDE              |
| 11 | N0610004 | BAKTERİ FİLTRESİ   |
| 12 | UN01D125 | ISITICI            |

### 3. ASSEMBLING DRAWING AND SPARE PARTS
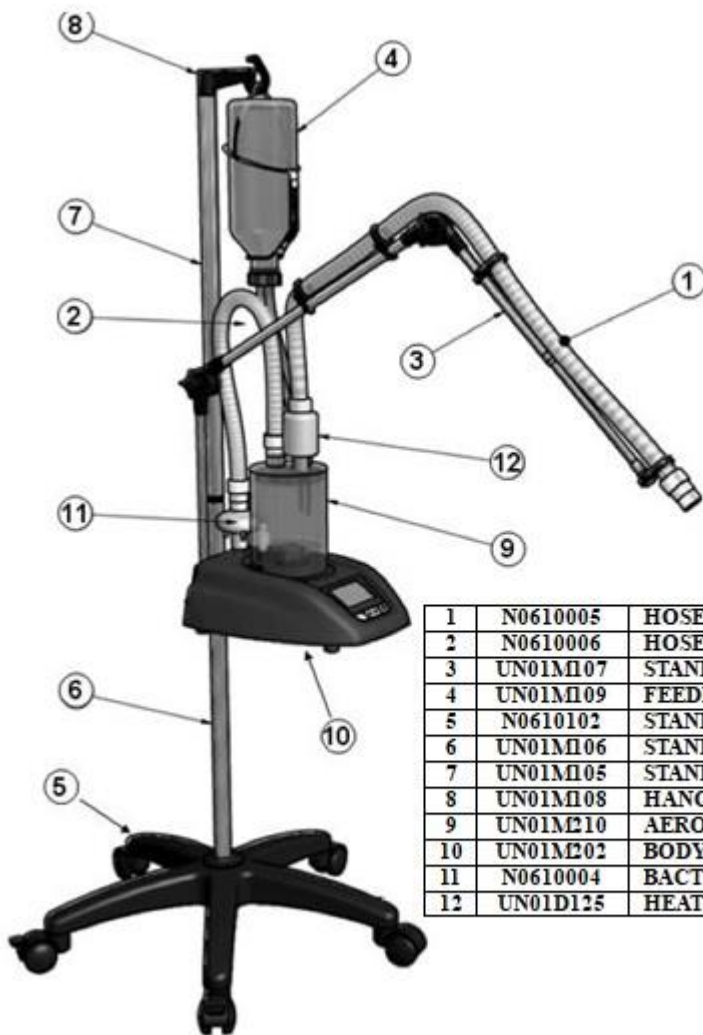
|    |          |                  |
|----|----------|------------------|
| 1  | N0610005 | HOSE Ø20x1200    |
| 2  | N0610006 | HOSE Ø20x450     |
| 3  | UN01M107 | STAND ARM        |
| 4  | UN01M109 | FEEDING BOTTLE   |
| 5  | N0610102 | STAND KIT        |
| 6  | UN01M106 | STAND PIPE LOWER |
| 7  | UN01M105 | STAND PIPE UPPER |
| 8  | UN01M108 | HANGER           |
| 9  | UN01M210 | AEROSOL JAR      |
| 10 | UN01M202 | BODY             |
| 11 | N0610004 | BACTERIUM FILTER |
| 12 | UN01D125 | HEATER           |

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## 4. INSTALLATION

After taking out the NEBTIME nebulizer device from its packing, for installation please follow below instructions. During installation, if you encounter any difficulty please contact with your authorized technical service personnel.

Install the casters by gently pressing down on the stand. Place the lower part (the part with the female thread on both sides with the small hole facing downwards) into the stand.

Place the flange part on the back of the stand leg.

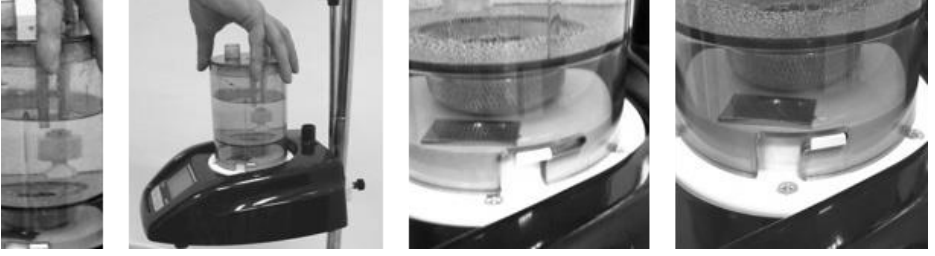
Tighten the M8 bolt with the help of an allen wrench to hold the flange and the lower part of the stand.

Place the device on the lower part of the stand by the rear handle on the body. After adjusting the height of the device from the ground, fix it with a butterfly screw.

Place the stand hanger so that it engages in the upper part.

Attach the stand arm to the rear handle of the device and fix it with the butterfly screw

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
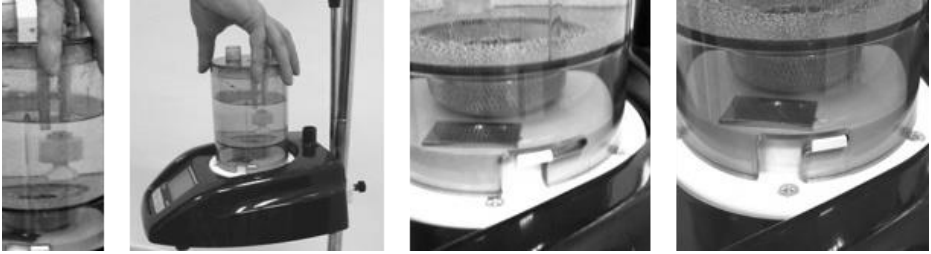
Fill the liquid jar up to the end of the bars on the upper lid (sterile water is advisable) dose the lid firmly and fix it on the device. lock on its place by turning the jar in the clockwise.

Place the bacterial filter on the air outlet of the device. Place one tip of the short hose to the outlet of the bacterial filter and the other tip to the outlet of the jar. If the heater is intended to be used heater will be mounted on the other outlet of the jar. Also, place the long steam hose on the outlet of the heater. If the heater is not intended to be used directly place the long hose to the outlet of the jar.

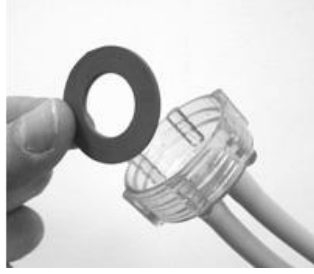Fix the socket of the heater cable to the male socket of the device located on the back side (if you experience difficulty while fixing turn the socket to the right and left side enable its location on place). Squeeze the bolt on the socket to the across side. Fit the long hose on the cuffs over the stand. Arrange position of the stand by loosening the butterfly screws on the joint components and squeeze again.

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Fix the bottle straps on the feeding bottle as shown in the drawing. Firmly fix the lid molded its bolt on the bottle. Fix the silicon hoses to the lid.

Firmly fix the bottle hoses into the small holes located on the jar lid. Fix the bottle sling into the stand sling keeping the bottle turned upside down. If the water flowing out from the bottle fills the jar until the highest level it means that the bottle lid is not fixed. Repeat the action by emptying the water in the jar. Liquid in the bottle carries out automatic feeding as the liquid level in the bottle is decreased. Liquid level in the jar remains same. Adjust the height of the device from the ground level by the butterfly screw on the back handle. You have completed the installation of the device and made it ready to operate.

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## 5. SAFETY

Read the guide book carefully before use.

It is recommended that you check that the electrical voltage of the socket to which you will connect the device's power cord corresponds to the operating voltage of the device.

During the transportation the electricity cable of the device must be taken out from the fuse.

Do not operate the device very close to heat sources.

Do not stick any substance into the air filter holes of the device.

Keep the device away from the children.

After operating the device, turn off on the turn on/off switch and take out the electricity cable from the fuse.

Do not pour water on the device.

In case of any breakdown read the BREAKDOWN section of this guide.

Except the liquid materials recommended by your doctor never use that device for evaporating any liquid.

## 6. OPERATION

If you attempt to use drugs on your device. After putting drug into its pot, firmly fix into the jar lid. Later on, fix the jar lid on the jar. If you do not attempt steam application fix the lid on the jar without the drug pot. Assemble the device where it is going to be used and lock compressing the wheel locks. Do not put the on tabletop models on the ground Adjust the spiral arm for the most optimum use.

Plug in the one end of the electrical cable to the device and the other end to the on/off switch on the back side of the device. Turn off from here when the device operation is over.

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When the device opens NEBTIME logo will appear on the screen.

If the system does not have any problems after checking itself you will see "STAND BY PRESS BUTTON" on the screen. If there is a problem with the device, it will give you a visual and audible warning.

For the operation of the device. 

Press on the STANDBY button 1 second. The device will start and the this screen will appear.

If the heater cable is not plugged in or if there is a problem with the heater while the cable is plugged in, HEATER DISCONNECTED will warn you as it is on the screen. However, this does not prevent your device from working, the device will continue to work.

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## 7 – CHANGING DEVICE SETTINGS

Menu button: Used to enter the setting menus of the functions. Saves the setting of the current function at each press and goes to the next function setting menu.

Level adjustment buttons: Right side increases level and left side decreases.

1. Adjustment menu: In order to change the settings, when the device is operating press once on the menu button you will enter to AEROSOL (steam power) adjustment menus. With the right-left buttons, the desired level can be adjusted between 1 - 10. press the menu button to exit and enter the 2nd adjustment menu.

2. Adjustment menu, AIRFLOW (air blowing intensity). With the right and left buttons desired level is adjusted in 1-10 range. Press the menu button again to exit and enter the 3rd adjustment menu.

3. Adjustment menu, TIMER. In 5-10- 15- 20-30- 40-50-60-90-120 minutes, it can be adjusted to the desired level with right-left buttons in the continuous operation mode range. Press the menu button again to exit and enter the 4th adjustment menu.

4. Adjustment menu is the HEATER (heater intensity). If the heater is installed and to be used, the desired level can be adjusted between 1 - 10 with right-left buttons. Press the menu button again to exit. The device returns to normal operation mode.

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The settings are automatically saved in memory and remain constant until changed by you. In normal operation mode, under the picture of the every function, the value it is set and how many minutes it will turn off can be seen

For example on the screen .

Steam 10, air 2, time length 120 minutes, heater 8 and remaining time are seen as 120 minutes.

The device switches off automatically after the set time and switches to STANDBY mode. To restart, simply press and hold the STANDBY button for 1 sec. The system restarts with the last settings. To put the running device on hold, you must press and hold the STANDBY button for 1 second.

WHEN THE DEVICE IS NOT IN USE TURN OFF BY PRESSING ON THE ON/OFF BUTTON ON THE BACK SIDE.

## 8. WARNING SYSTEM

Heater warning: If the heater is not used and you have not fixed the heater cable to the socket located on the back side, normal operation mode will give a visual warning as HEATER DISCONNECTED. But no acoustic warning will emerge. The device will keep operating without any matter.

Liquid level warning: If the liquid jar is not firmly fixed and liquid level in the jar decreased to the minimum level system will give visual warning(PLEASE ADD LIQUID INTO THE CHAMBER)as on the screen, acoustic alarm will ring and device will stop. Place the jar firmly in place and add liquid to the normal level.

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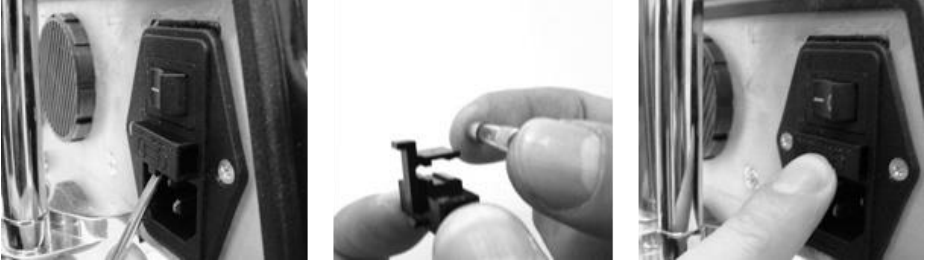
## 9. MAINTANENCE

### General care:

The device should be checked at least once a year by authorized technical staff.

### Air filters replacement:

Unlock the air filter cover by turning it clockwise and pull it toward you to remove it. Replace the filter with a new one. Lock it again by turning it counterclockwise.

### Replenishment of the fuses:

If your device is not working completely.

Make sure that the power cord is correctly connected at both ends. Your device's on-off switch is on. Check that there is electricity in the socket to which the device is connected. If there is no problem, the fuse of your device may be blown.

**Warning:** Before changing the fuse, the power cord of the device must be disconnected from the device. The fuse must be replaced by a qualified technician.

**Replacement:** There is a fuse holder on the plug socket on the back of the device. Open it by pulling it towards you with the help of a thin screwdriver. Replace the inside glass fuse with a new one (1.6 A - 250 V) (new devices have 1 spare fuse in the fuse holder). Refit the fuse holder and push it forward.

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### **Replenishment of the bacterial filter :**

Bacterial filter is used in order to filter the air absorbed from the environment which given to the patient with the steam. Therefore, after a certain period of use must be replaced with a new one.

## **10. CLEANING AND DISINFECTION**

Jar and lid: Autoclave 121C-20mins recommended

Air and steam hose: After every use Autoclave 121C-20mins recommended

Devicebodyandthestand spareparts: Wipe with a cloth soaked in soapy water and thoroughly wrung

## **11. TECHNICAL PROPERTIES**

|                       |                              |
|-----------------------|------------------------------|
| VOLTAGE               | 230 VAC $\pm$ % 10 , 50 Hz   |
| POWER                 | 40 W - 70 W ( WITH HEATER )  |
| ELECTRICAL TYPE/CLASS | CLASS I , TYPE BF            |
| OSCILLATORFREQUENCY   | 1,68 MHz $\pm$ % 5           |
| PARTICLE SIZE         | 0,5 - 6 $\mu$ m              |
| NEBULIZATION CAPACITY | 0 - 5 ml / min $\pm$ % 10    |
| AEROSOL TANK          | MAX. 750 ml                  |
| WEIGHT                | < 4 KG                       |
| AEROSOL TEMPERATURE   | MAX . + 38 ° C (WITH HEATER) |
| OPERATING TEMPERATURE | + 10 $\sim$ + 30 ° C         |
| OPERATING MODE        | CONTINUOUS                   |

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## 12. BREAKDOWN

If the recommended practices do not help you to rectify your fault, please contact the authorized technical service.

If you encounter any other faults not listed in this table, contact the authorized technical service.

| BREAKDOWN  | SOLUTION   |
|--|--|
| “HEATER DISCONNECTED” ON THE SCREEN                | SEE: WARNING SYSTEM PAGE 21  |
| “PLEASE ADD LIQUID INTO THE CHAMBER” ON THE SCREEN | SEE: WARNING SYSTEM PAGE 21  |
| THE DEVICE IS NOT OPERATING                        | SEE: CARE- FUSE REPLACEMENT PAGE 23  |
| EXCESS WATER FILLED IN THE JAR                     | CHECK THE FEEDING BOTTLE LID. SEE: INSTALLATION FEEDING BOTTLE ASSEMBLY, PAGE 13   |
| NO AUTOMATIC WATER SUPPLY TO THE JAR               | HOSES BETWEEN THE FEEDING BOTTLE AND THE JAR MAY HAVE BROKEN PLEASE CHECK  |
| THE DEVICE IS OPERATING WITHOUT GIVING ANY STEAM   | CHECK THE WATER MAY HAVE ACCUMULATED IN THE HOSE<br>CHECK THE FAN MAY NOT PROVIDE AIR<br>CHECK HOSES MAY BE BROKEN OR CRUSHED<br>CHECK THE WATER LEVEL IN THE JAR CAN BE VERY HIGH.<br>PESOCERAMIC CRYSTAL MAY BE DEFECTIVE AND REPLACE WITH NEW |

## 13. ACCESSORIES

Medicine cup  
Air tubing  
Patient tubing  
Feeding bottle  
Bacterial filter  
Aerosol cup  
Flexiable arm

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## 14. GUARANTEE

1. Warranty period (2 years) starts from the date of delivery of the goods to customers.
2. If the device fails within warranty period, the time spent in repairing is added to the warranty period. The repair period is maximum 30 working days
3. If you have a complaint, please send the complete device to your specialist dealer or send it to us in the original packaging, postage paid including warranty certificate inside with the specialist dealers stamp.
4. Damages arising from improper use of the device are not covered by this warranty
5. This warranty will be invalidated if the device is interfered with by unauthorised persons
6. Malfunction from not following the instructions in the manual are not covered by the warranty.
7. This warranty is valid only if all the conditions stated herein are read and accepted.
8. Consumable parts are not covered under warranty

## 15.SERTİFİKALAR / CERTIFICATES

ISO 9001

ISO 13485

CE

## 16. ÜRETİCİ FİRMA / MANUFACTURER

### ELMASLAR

İmalat Tibbi Cihazlar İthalat İhracat A.Ş.

ASO 1. Org. San. Bölgesi Babürşah Cad. No: 17

Sincan /ANKARA / TURKIYE

Tel: +90 312 394 80 01 Fax: +90 312 394 80 04

info@elmaslarmedikal.com.tr • elmaslarmedical@gmail.com

www.elmaslarmedikal.com.tr

## **ELMASLAR A.S.**

ASO 1.OSB BABURSAH CD. NO: 17 SINCAN / ANKARA / TURKEY

Tel: +90 312 394 80 01 Fax: +90 312 394 80 04

info@elmaslarmedikal.com.tr • elmaslarmedical@gmail.com

[www.elmaslarmedikal.com.tr](http://www.elmaslarmedikal.com.tr)