**INSTRUCTION ON 7E-D AC/DC
ABSORB PHLEGM UNIT**

7E-D AC/DC ASPİRASYON CİHAZI

Please carefully read the instructions

before attempting to operate this unit.

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FEATURE ÖZELLİKLER

I Summary - Özet

Model 7E-D AC/DC electric suction apparatus is a movable and portable pressure suction device, which is designed on the basis of advantages of international products. It is suitable for giving emergency treatment to a respiratory tract patient in hospital, first-aid teams, home care or social medical services. It is able to clean secretion, blood vomitus by which treat blocked so that breathe unobstructed especially available for patient first-aid and health care patient transport and place without AC/DC.

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II Characters - Özellikler

-oil-less piston driver vacuum pump assures steam free and lubrication free that prevents bacteria contamination.

-Overflow device prevent liquid entering pump.

-Plastic panel;

-Lower noise, massive flux, hand-switch and foot-switch for operating easily.

-Negative pressure adjustment system can be adjusted according to demands.

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Examine exterior of the unit for nicks, dents scratches or other damages, check for kinds and quantities to the accessories before installing and debugging. If damage is evident, if accessories have any doubt, please notify the local dealer or manufacturer.

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Note: Apply small amount of distilled water around the part (pressed into the holder mouth) of holder plug during installing, which is good for tightly pressing the holder plug and enhancing its sealing.

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Caution: The blue face on the air filter is air entrance, which should connect with exit of store-liquid bottle.

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Draw back handle, and take out power thread, then connect power.

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Caution: Power plug can cut off power. The power socket must connect grounding.

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- Tighten adjustment valve, and block up air entrance with fingers or rubbers, then pinch soft tube.
- Starting the machine, without allophone, the needle on the vacuum meter rise to utmost pressure immediately. Unloosening air entrance, the needle will be below 0.02MPa. That is to say that tube connections are right.
- Attach the phlegm suction catheter. The vacuum value shall be less than 0.06 MPa at the time of attaching F6 suction catheter, less than 0.04 MPa when attaching F8 suction catheter and less than 0.03 MPa when attaching F12 suction catheter. If so, the phlegm suction is considered as being in normal condition.
- Ayar valfini sıkın ve hava girişini parmakla veya plastik bir parçayla tıkayın

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Note: Dredge the suction catheter if blocked as' per the following method: Bend the suction conductor in "V" form (with no liquid in the holder), and release it to the original status when the vacuum reaches up to the maximum value. Repeat this procedure several times till the suction catheter is not blocked.

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Block up air entrance, switch on suction apparatus, and adjust negative pressure adjustment valve. The readings of pressure meter are changed in the range 0.02MPa to utmost pressure.

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- **Control pressure with adjustment valve in clinic.**
- **Pressure changes through circumvolving adjustment valve.**
- **Reduce pressure below 0.02MPa before switching off the machine.**

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V **Examine, try overfall device** - Deneme

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Caution - Dikkat:

After switching off the overfall device, the liquid site still rise, has two reasons:

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One reason is from the remainder negative pressure in the store-liquid bottle.

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The valve mouth is incomplete closed up. Go through the reason of the valve mouth's invalidation.

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When valve is shut off by buoy, the buoy could still be absorbed on the valve mouth.

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The reason is negative pressure on the tube road. At this time loosen adjustment valve or switch off suction apparatus to let out negative on the tube road. Under the effect of gravity, buoy will fall from valve mouth.(Strictly prohibit pulling buoy with hand to avoid valve slide come over with buoy.)

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After stopping the machine, let out negative pressure, then open the bottle plug.

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After taking down overfall device and tubes, prohibit using the suction apparatus.

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VI Stop machine – Cihazı Durdurmak

Switch off the suction apparatus first, then pull out plug-and-socket, and cut off net power supply.

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Symbol and signification

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Symbol	Signification	Symbol	Signification
~	Alternating current		Note! Refer to the manual
	Protection earthing		

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Please check the internal battery if is charged fully before using. Lütfen

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Connect the power cord to a properly grounded AC outlet, the charging light is bright shows the power supply and charges internal battery. The charging light gleams to mean to battery charging fully.

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Break net power, then use the internal battery.

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The internal battery shall be charged fully about 4 hours when it was been used up (the red light bright). The internal used battery is charged according to residual power capacity in order to power capacity fully.

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The internal battery shall be charged and discharged once a month to maintain if it was not use for a long time.

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Note : The unit has internal charger and shall be not use any other specification battery. The battery shall be not used over 50 minutes once.

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If dirt is haled into air filter, the color of filter velum will turn from light to deep, even induce suction power reduce, even disappear. Negative pressure on the pressure meter rise beyond 0.04MPa, at this time exchange air filter in time. Air filter should exchange frequently, and destroy together.

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III Exchange of fuse tube Fuse tpn deęiřtirilmesi

Fuse tube is fixed at the back of the pedestal. Before exchanging it, switch off power first.

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OTHERS ATTENTION ITEMS

DİKKAT EDİLECEK DİĐER KONULAR

I Limitary conditions of transportation and store environment

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- **Environment temperature range: -40°C ~ 55°C**
- **Relative humidity range: ≤95%**
- **Air pressure range: 0.050MPa ~ 0.106MPa**
- evre sıcaklık aralıęı -40°C ~ 55°C
- Nem Aralıęı ≤95%
- Hava basın aralıęı: 0.050MPa ~ 0.106MPa

Suction apparatus should place in erosive less gas or ventilation room, and avoid acute shock when transported.

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